

Quickstart manual

1 Minimum Requirements

The device connects to a computer with at least the following characteristics:

- Windows 10 or 11, or MacOS Monterey (version 12) or later
- Intel i3 for Windows versions
- Intel i5 or M1 chip for MacOS versions
- 8 GB RAM
- Recommended resolution: at least 1600*900
- To use a Bluetooth accessory (BioMoov, BlueBack, Emy or Perifit), you need a PC running **Windows 10 or 11** equipped with a Bluetooth card, or MacOS

Please note that Biostim software **does not work** with :

- Windows 10S or 11S
- PCs with an ARM chip
- Tablets or smartphones (Android or iOS)
- Chromebooks

2 Software requirements

The following software must be installed on your computer:


- FTDI Driver (installation via CDM212xxx_Setup.exe supplied with software)
- Foxit PDF Reader

If they are not already present on the computer, the installation files are available on the key in the "drivers" directory.

3 Installation

3.1 Windows

Install the program on the desktop by double-clicking on the **installation** file (or **installation.cmd**) available on the provided USB stick.

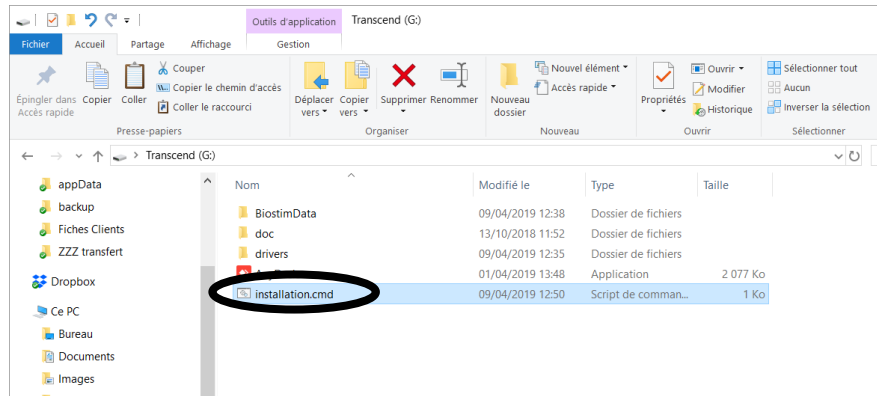

This operation creates a BiostimData folder in the user directory (which will contain all patient data), as well as a shortcut on the desktop.

3.2 Mac OS

Run the Biostim_Installer.pkg program.

This operation creates a BiostimData folder in the user directory (which will contain all patient data), as well as a shortcut on the desktop.

4 Start

Set the ON/OFF switch on the rear of the unit to ON "1".
Check that the green power indicator light on the front of the unit is on.

Run the Biostim program on your computer



5 Connection check

Check that connection is established: green home button.



The **red** home button indicates a communication problem between the computer and the device. In this case, check the following points:


- The module is powered up, the green LED on the front panel is lit.
- The USB cable is properly connected to both the device and the computer.
- FTDI driver correctly installed (CDM212xxx_Setup.exe)

The **orange** home button indicates a problem with the emergency stop bulb:

- Check that the emergency stop bulb is connected
- If the emergency stop has been activated, the BioStim must be restarted (on/off switch at the back of the unit).

6 In the event of a problem

Go to the Autotest tab. If a problem is detected, it will be indicated in red. If there is no internet connection, the autotest logo will be red.

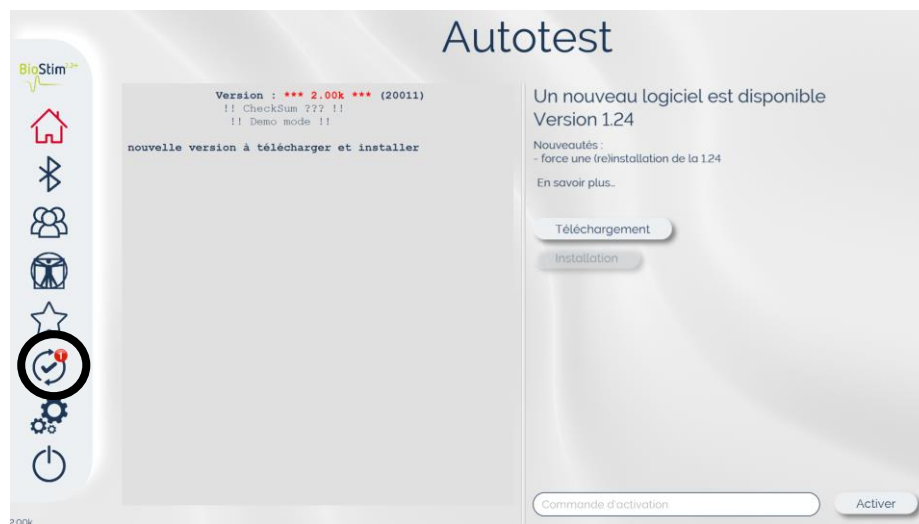

If you need more assistance, go to the Contact tab on the configuration page, where you can report problems or make suggestions by e-mail.



7 Remote control

During installation, AnyDesk software will be installed on the PC. It enables us to take remote control of the PC to manage an after-sales service.

A shortcut is available on the



To authorize a technician to take control, you need to give him the login and password that appear in the window after launching the software.

