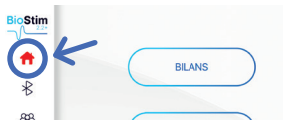
5 Vérification de la connexion Minimum configuration requirements

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• Check the home button to verify the connection status:

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Blue: The connection is established and operational.

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Red: Communication issue between the computer and the device. In this case, check the following:

1. The device is powered on (the green indicator light on the front should be lit).
2. The USB cable is securely connected to both the device and the computer.
3. The FTDI driver is correctly installed (using the *CDM212xxx_Setup.exe* file).

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Orange: Issue with the emergency stop button:

1. Ensure the emergency stop button is properly connected.
2. If the emergency stop was activated, restart the BioStim device using the power switch located at the back.

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• Click on the « Autotest » button, located in the side menu: 
- If a problem is detected, it will be displayed in red. Follow the provided instructions to resolve it.
- If there is no Internet connection, the « Autotest » logo will turn red.

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
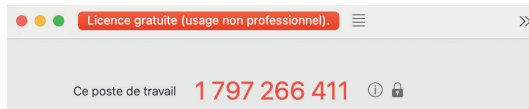
7 Assistance à distance Remote support

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• During installation, the AnyDesk software was installed on your computer. This software allows a technician to remotely access your system for technical support.
A shortcut is available on your desktop for easy access. To authorize a technician to connect, launch AnyDesk and provide them with the ID and password displayed in the software window.

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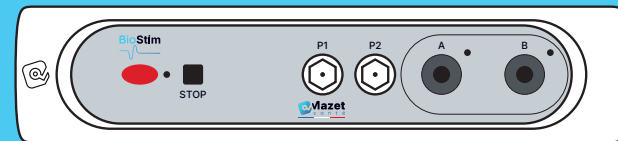- The device connects to a computer with at least the following specifications:
 - **Operating System**: Windows 10 or 11, or macOS Monterey version 12) or later
 - **Processor**: Intel i3 for Windows versions / Intel i5 or M1 chip for macOS versions
 - **RAM**: 8 GB
 - **Recommended Resolution**: At least 1 600×900

For using a Bluetooth accessory (BioPod, BioMoov, BlueBack, Emy, or Perifit), a PC running Windows 10 or 11 with a Bluetooth card, or macOS, is required.

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IMPORTANT : The BioStim software does not work with:

- Windows 10S or 11S
- PCs with ARM chips
- Tablets or smartphones (Android or iOS)
- Chromebooks

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- The following software must be installed on the computer:
 - **FTDI Driver** (installed via the **CDM212xxx_Setup.exe** file, provided with the software)
 - **Foxit PDF Reader**

If this software is not already installed, the installation files are available on the USB drive in the « **drivers** » folder.

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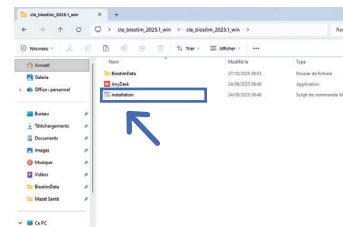
- To install the program, double-click the « Installation » utility (or « installation.cmd »), located at the root of the USB drive.

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
- This process creates:
 - A «**BioStimData**» folder in the user directory (to store all patient data)
 - A desktop shortcut for quick access.

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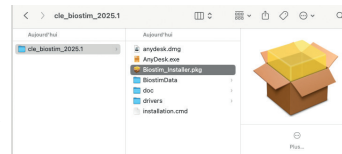- Run the « Biostim_Installer.pkg » program, located at the root of the USB drive.

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- 1. Turn the power switch, located at the back of the device, to the ON « 1 » position.
2. Ensure that the green power indicator light turns on at the front of the device.
3. Launch the Biostim program on the computer.