

BioStim Version 2026.1

BioPod



This new version is compatible with BioPod Full.


Unlike the BioPod Link, the BioPod Full does not need to be connected to a BioStim to function.

It is the ideal tool for performing BFB independently.

More information:
<https://www.mazetsante.fr/biopod/>

To connect, click on the Bluetooth logo. Once connected:

- A BioPod logo will appear next to the Bluetooth logo.
- The Biopod channels will appear in the list of available channels

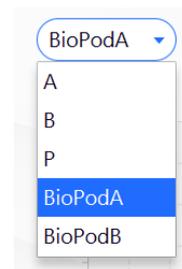

Available programs

Programs are no longer immediately available when the software is installed. To display them, simply connect a BioStim or Biopod Full. Programs compatible with your devices will be automatically unlocked


Changing the default program times



The default program time (20 minutes in BFB and 15 minutes in stimulation) can be adjusted without modifying each program individually.


BioMoov

There are now 4 options available for choosing the default behavior when a BioMoov is connected:

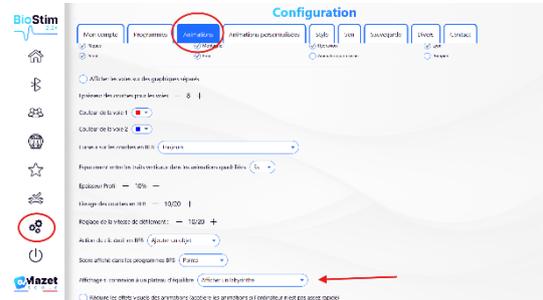


- Do not display anything
- Display a neutral plateau
- Display a tray with exercises
- Display a maze
- Use a goniometer in channel 2

In the "Target" programs, you can choose the size of the target to be hit in order to adjust the difficulty:

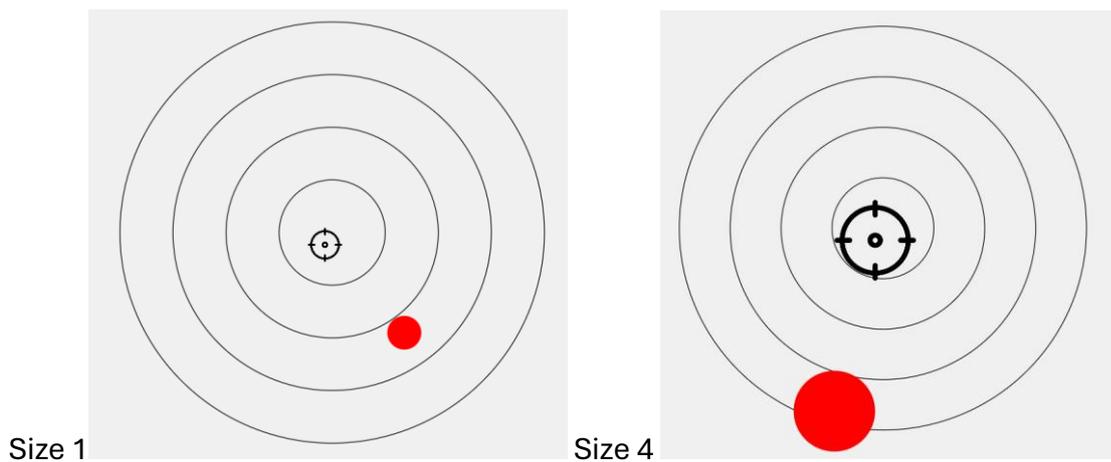


New "Valentine's Day" animation

A new Valentine's Day animation is available

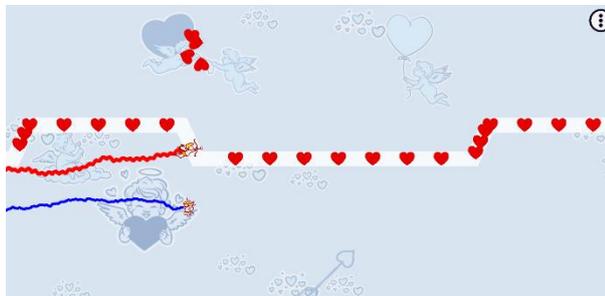


Choice of cursor

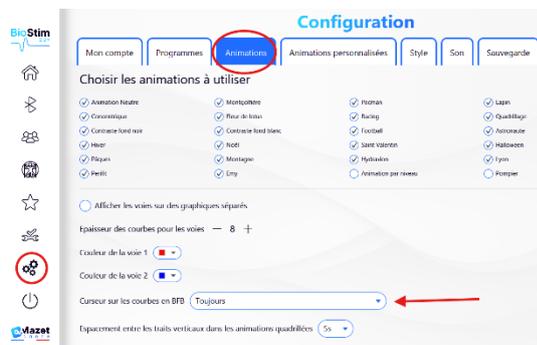


You can choose the type of cursor you want for the animations:


None

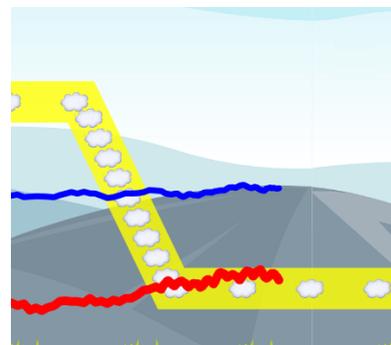

Triangles

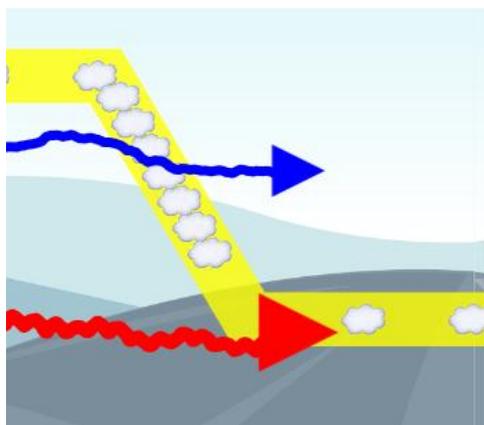


Animation cursors

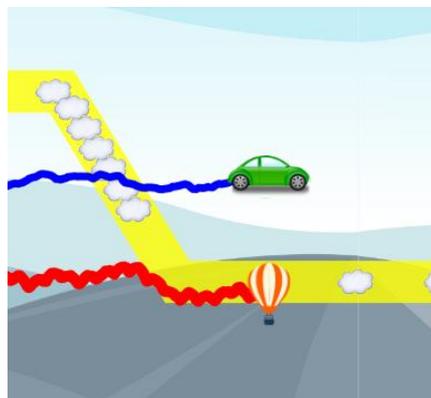

Random objects in relaxation

In BFB Relaxation, if you choose to display random objects, they appear at the maximum relaxation level reached by the patient.

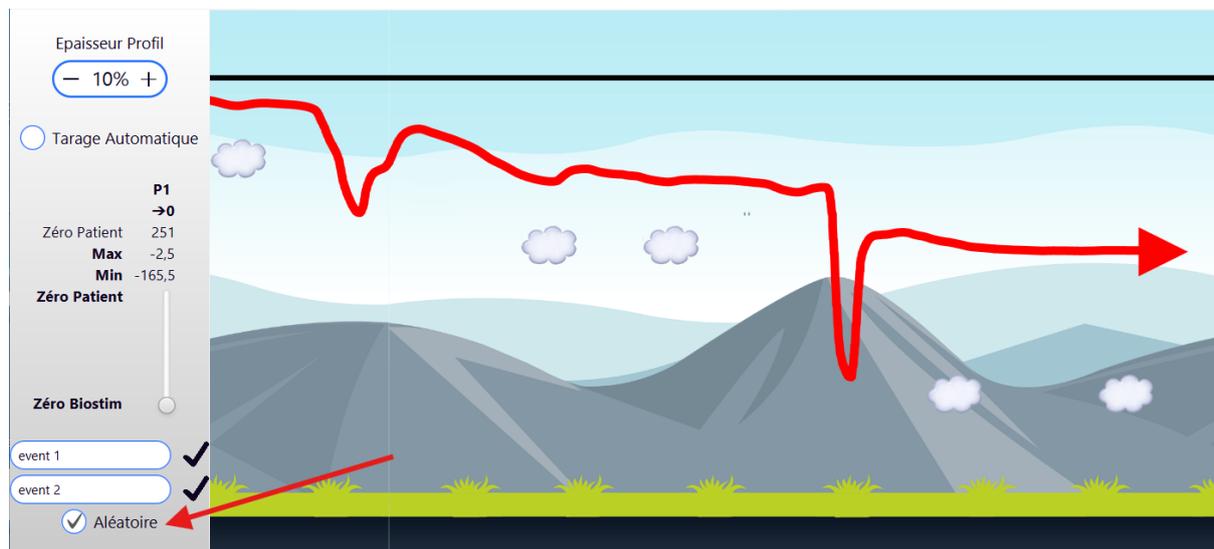

Miscellaneous and bug fixes

- Added tooltips to certain buttons, in particular the "Archive" button in the patient list
- Fixed: Videos were no longer displayed in the anatomy charts following a change to YouTube's rules
- Improved the way contractions are counted in the "fast fibers" section of the reports.

Update: **Please note!**

- Updating versions older than 2025.2 is a two-step process: update to version 2025.2, then update to 2026.1. Everything is automatic, but you must perform two updates in succession.
- PCs running Windows 7 and 8, as well as Macs running a version of MacOS earlier than 12, are not supported. No updates will be offered for them.