



Computer choice for the Biostim

All versions

Compatible computers

The Biostim is compatible with:

- Windows 8, 11 or 11 PCs, desktop or laptop
- Macs, desktop or laptop

The Biostim does not work with:

- Tablets or smartphones regardless of the system (Android, iOS)
- Chromebook

Special Case: Windows 10S is not compatible, but Microsoft allows to activate a 10S version for free to transform it into a "classic" version 10, therefore compatible.

Basic requirements

Memory: 4GB

Processor:

- Intel i3 for Windows versions
- Intel i5 or M1 chip for MAC versions

Display: the device works on any type of screen, but:

- The use is not recommended on screens of 14 inches or less
- The minimum resolution is 1366*768 (HD), it is recommended to use at least 1600*900 (HD+)

The connection with the device uses a USB port. Check that there are enough ports if you want to connect an external mouse or keyboard.

BlueTooth accessories (Blueback / BioMoov / Emy / Perifit)

Bluetooth accessories only work with Windows 10 or 11 PCs (not Macs).

The computer must also have a Bluetooth Low Energy device. If this is not the case, it is possible to buy one to plug into the USB.

Others

Avoid purchasing replacement power supplies that are not from the computer manufacturer. The "compatible" versions often generate disturbances which degrade the biofeedback signal.